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REF These instructions for use apply to all spectacle frames manufactured by Dominik Gasser Holzbrillen.

1. Introduction

Thank you for choosing to purchase DG wooden spectacles.

- Quality with real roots
- Swiss craftsmanship

2. Intended use / purpose

The spectacle frame is designed to hold corrective lenses securely and should be worn in accordance with the general instructions for use for spectacle frames.

3. Contraindications, undesirable side effects

Wood, buffalo horn and the varnish used are often used for spectacle frames and are generally well tolerated. The metal parts do not come into direct contact with the skin. Nevertheless, attention should be paid to compatibility in the case of specific allergies.

4. Warnings

- Do not expose the glasses to extreme conditions, such as saunas, storage on the car dashboard, strong sunlight or intense sporting activities, contact with strong cleaning agents and chemicals. Cleaning in an ultrasonic bath is not recommended.
- Never clean glasses with alcohol or solvent-based cleaning agents (perfumes and cosmetics also often contain alcohol or chemical substances that can negatively affect the lifespan of your glasses).
- The spectacle frame may become deformed in the event of a strong impact and cause injury.
- Allergies: Wood, buffalo horn and the varnish used are generally well tolerated, but if you have specific allergies, you should pay attention to the materials used.
- **Buffalo horn temple tips:** Dogs really like the taste of buffalo horn, so make sure your dog doesn't mistake the buffalo horn tips of your glasses for a chew toy.

5. Further information

- Individual adjustment: As with all spectacle frames, your optician should adjust the glasses to the shape of the wearer's face.
- Careful care and handling of glasses can significantly extend their service life.

6. Cleaning and care

- **Cleaning the lenses:** Spray 1-2 squirts of lens cleaner onto a microfibre cloth and clean the lenses. Then dry the lenses with a dry microfibre cloth. Make sure that there is no sand, dried-on limescale or salt on the glasses, as these substances can damage your lenses.
- **Cleaning the spectacle frames:** do not rinse the spectacles too often (once a month) with washing-up liquid under lukewarm tap water and dry them immediately with a clean paper towel or cotton cloth.

7. Storage – protection against damage

- Store your glasses in a sturdy case when not in use. This protects them from knocks, scratches and other damage.
- Make sure that the glasses are not squashed between other objects, as this can lead to deformation, breakage or other damage.

8. Inspection and repair

- Check the frames regularly for possible damage such as cracks, breaks, deformation or loose connections. If you notice any such problems, have your glasses checked by your optician immediately.
- If the spectacle frames show signs of wear or are damaged (e.g. white spots, discolouration, dents or breaks), it is advisable to send them to the manufacturer for repair via your optician. Do not attempt to repair the spectacles yourself, as improper repairs can further damage the spectacle frames.

9. Disposal

If the glasses are irreparably damaged, they should be disposed of properly (take them to a recycling centre or your optician).

10. Compliance

Our spectacle frames have been manufactured and tested in accordance with the EN ISO 12870:2018 standard "Spectacle frames – Requirements and test methods" and meet the requirements of the Medical Devices Regulation (EU) 2017/745. A corresponding EU declaration of conformity is available.

11. Contact person for questions or uncertainties

If you have any questions or concerns about the product or its use, please ask your optician.

12. Reporting to the manufacturer and the competent authority

All serious incidents related to the product must be reported to the manufacturer and the competent authority of the Member State in which the user and / or patient is located.

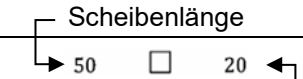
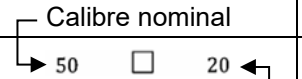
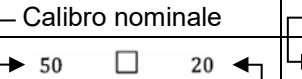
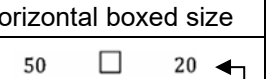
Erläuterung der Symbole / Explication des symboles / Spiegazione dei simboli / Explanation of the symbols

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The symbols on the frame and on the packaging are explained below.

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